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Producent:
KGK TRADE sp. z o.o. sp. k.
Ujastek 5B,
31-752 Kraków, Polska

nenó[®]
www.neno.pl
inspired by Children,
designed by Parents
Wyprodukowano w P.R.C.

USER'S MANUAL

Dear Customer,
thank you for purchasing Neno Leite, a silicone milk collector for collecting milk during feeding / pumping from the other breast. It aims to collect food in a simple and effective way.

01. SAFETY MEASURES

1. Before each use, check the condition of the product.
2. We recommend using only soft brushes or sponges for cleaning, hard materials may scratch.
3. If there is any sign of damage, replace the product.
4. Do not store near sharp objects.
5. This product is not a toy. The collector should be used in accordance with its intended use.
6. Store the product in a dry place and avoid direct sunlight

02. PRODUCT FEATURES

Easy to use
With just one squeeze, the milk collector draws out your milk with the power of continuous suction-no noise, no electricity and no cords. Pump anywhere, anytime!

Easy to clean
All-in-one design and no disassemble required. They are very easy to clean by direct boiling water for about 2 minutes.

BPA FREE Material
Made of high quality of silicone. No BPA, PCV and phthalate.

Painless and work silent
The milk collector helps to express breast milk discreetly, silently and quickly. Avoid embarrassment in public places.

No Leak Any More
When most mothers are breastfeeding, the breast on the other side would often spill breast milk which may wet your clothes or your baby, or be wasted on the disposable nursing pad. The milk collector can collect every drop of breast milk while you're breastfeeding.

03. HOW TO USE

1. Make sure your milk collector has been cleaned before and after use.
2. Squeeze the pump to push internal air out.
3. Invert the flange, place it onto your nipple centered in the middle of the breast shield.
4. Restore the flange, then the pump would firmly suck onto your breast.
5. You may use the product as a collector. The milk will then flow freely to the pump. If you want to use it as a manual breast pump - squeeze the pump to facilitate milk leak.

04. SPECIFICATION

Material: Silicone+PP
Capacity: 100 ml

05. WARRANTY CARD

The product is covered by a 24-month warranty.
The terms of the guarantee can be found at: <https://neno.pl/gwarancja>

Details, contact and website address can be found at: <https://neno.pl/kontakt>

Specifications and contents of the kit are subject to change without notice. We apologise for any inconvenience.

Manufacturer:
KGK TRADE sp. z o.o. sp. k.
Ujastek 5B,
31-752 Cracow,
Poland

nenó[®]
www.neno.pl
inspired by Children,
designed by Parents
Made in P.R.C.

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